ventura impress 2 putty soft

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EN INSTRUCTIONS FOR USE

Ventura Impress 2 is a system of materials for high precision dental impressions based on addition curing silicones (polyvinylsiloxane), particularly indicated for the double impression technique.

Both the moldable putty, **Ventura Impress 2 Putty Soft**, as well as the fluid paste **Ventura Impress 2 Light Body**, packaged in cartridges, possess, once polymerized, excellent mechanical properties and a great capacity for reproducing details.

Handling is quite simple, since in all cases the base putty and the catalyst have the same density, and the mixture is always made with a 1:1 ratio (by weight or by volume). The different colors of the two pastes provide good control of the mixture. The use of the fluid paste in cartridges greatly simplifies the procedure, producing homogeneous mixtures with a total absence of bubbles and a notable economy in material usage.

The hydrocompatibility of **Ventura Impress 2 Putty Soft** allows it to enter into close contact with humid surfaces without absorbing liquid, making possible a more perfect reproduction of details, particularly on the edges of the preparation.

This property also makes possible a better gypsum flow over the surface when preparing the model, penetrating in even the smallest cracks.

The polymerization of the polyvinylsiloxane does not produce volatile secondary components. This gives it excellent dimensional stability, which permits the preparation of the model even several days after taking the impression, without clinically notable alterations.

INSTRUCTIONS FOR USE

Draw equal amounts of the putty and the catalyst paste from their respective containers, using the corresponding measuring spoons for each component.

With fingertips, mix both components until a smooth paste with uniform color is obtained (approximately 30 seconds).

Use vinyl gloves, because the powder that protects most latex gloves can inhibit polymerization of the silicone.

Close the containers immediately following their use. Make sure to replace the correct covers and measuring spoons for each container. If the ingredients are handled separately before mixing, the setting time will be shorter.

MAKING THE IMPRESSION

Once the mixture of **Ventura Impress 2 Putty Soft** is completed, place the mixed material in the tray (can be metal or plastic, with or without perforations).

Within 1 minute from starting the mix, place the tray with the material in the mouth and keep it in position for at least 1 minute and 30 seconds (fast set) or 2 minutes and 30 seconds (regular set).

MODELS

Models can be poured after 30 minutes, and within seven days of polymerization. The impression should be kept at room temperature until the moment of removal.

Ventura Impress 2 Putty Soft is compatible with all quality gypsums, such as **Fuji Rock**, **Whip Mix** or **Ventura**.

ELECTROPLATING (GALVANIZATION)

Ventura Impress 2 Putty Soft impressions can be galvanized by acid baths of copper or silver alkalines.

PRECAUTIONS

In sensitive subjects, vinyl-polysiloxanes may cause irritation or other allergic reactions.

Contact with latex gloves may prevent vinyl-polysiloxane from setting. Avoid direct or indirect contact with latex gloves. Instead, wash your hands thoroughly to eliminate all traces of impurities or use vinyl gloves.

Silicone adheres to clothes from where it may be very difficult to remove, even by using chemical agents.

STORAGE

In order for the product to maintain its optimum characteristics, it is advisable to store it at temperatures between 10°C and 23°C and use it before the expiry date.

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TECHNICAL CHARACTERISTICS	Putty Soft Regular Set	Putty Soft Fast Set
Type	: Moldable putty	: Moldable putty
Classification ISO 4823	: Very high viscosity - Type 0	: Very high viscosity - Type 0
Color	: Yellow	: Blue
Mixing ratio base : catalyst	: 1 : 1	: 1 : 1
Mixing time	: 30 seconds	: 30 seconds
Working time at 23 °C	: 2 minutes	: 1 minute 15 sec.
Setting time	: 3 minutes	: 2 minutes
Compression deformation	: approx.3 %	: approx.3 %
Recovery following deformation	: 99.5 %	: 99.5 %
Linear dimensional variation at 24 hours	: < 0.20 %	: < 0.20 %
Shore A Hardness at 24 hours	: approx. 60	: approx. 60

Instructions for use issued in January 2015 (2015-01)

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